

# Let us know

## Vaccine Storage in an ILR



### If the basket in the ILR is fully filled

Arrange two layers of empty ice packs at bottom of the ILR and keep OPV and Measles vaccines first on top of the empty ice packs. Then store the other vaccines as depicted in the picture.

### If there is no basket in the ILR

- Arrange two layers of empty ice packs at bottom of the ILR and keep OPV and Measles vaccines on top of the empty ice packs.
- Store all the other vaccines as depicted in the picture.
- Keep one thermometer among the vaccines.
- Do not keep the vaccines in direct contact with the walls of the ILR

